

TML/MSH Microbiology Department Policy & Procedure Manual	Policy # MI/EDU/04/01/v01	Page 1 of 1
Section: Education Manual	Subject Title: Appendix I - Day One	
Issued by: LABORATORY MANAGER	Original Date: June 18, 2001	
Approved by: Laboratory Director	Revision Date:	

APPENDIX I - DAY ONE

1. On Day 1, subculture *S. aureus*, CNST, *Enterococcus* spp., *E. coli* and *Pseudomonas aeruginosa* onto MacConkey and Blood agar plates for isolated colonies. (See Appendix II).
2. On the second day gram stain each organism from any one plate. (See Appendix III). Also describe colony morphology for each organism on each of the plates.
3. On the second day also do the following tests and record results: (See Appendix IV).
 - Catalase - For either *Staph. Spp* and *Enterococcus* spp.
 - Coagulase - For *S. aureus* and CNST
 - Bile esculin - For *Enterococcus* spp.
 - MUG - For *E. coli*
 - Oxidase - For *Pseudomonas aeruginosa* and *E. coli*