MSH/TML Shared Microbiology Service Policy & Procedure Manual	Policy # MI\PAR\06\06\v01	Page 1 of 3
Section: Parasitology Manual	Subject Title: Collection and Laboratory Procedures for Specimens Other Than Stool or Blood	
Issued by: LABORATORY MANAGER	Original Date: March 13, 2000	
Approved by: Laboratory Director	Revision Date:	

Lung and Liver Aspirates

PRINCIPLE

Aspirates will frequently detect parasites when other methods fail. Organisms that may be found in aspirates include *P. carinii* and *E. histolytica*, from lung and liver respectively.

SPECIMEN

If an amoebic abscess or visceral leishmaniasis is suspected, the aspirate should be placed in a clean and empty bottle and submitted to the laboratory as soon as possible. As the amoebae are generally located near the wall or edge of a liver abscess, it is preferable to place the last part of the aspirate (frequently the reddish "anchovy paste") in a separate container since it is more likely to contain *E. histolytica*.

REAGENTS

Robinson's Medium (see appendix)

PROCEDURE

This is considered to be a non-routine procedure therefore it should only be performed by experienced personnel.

- 1. Start by examining the last part of the aspirate.
- 2. Examine the sample as a wet mount, or alternatively you can inoculate Robinson's culture media to grow E. histolytica and prepare a permanent stain slide.

QUALITY CONTROL

- Ensure that culture media is not contaminated.
- Ensure that the microscope has been calibrated in the last year and that the results of the calibration are displayed on the microscope base.
- See QC comments associated with haematoxylin staining.

PROCEDURE MANUAL

MOUNT SINAI HOSPITAL/TORONTO MEDICAL LABORATORIES SHARED MICROBIOLOGY SERVICE

MSH/TML Shared Microbiology Service	Policy # MI\PAR\06\06\v01	Page 2 of 2
Policy & Procedure Manual		
Parasitology Manual		

REPORT

The presence of any parasites.

LIMITATIONS OF PROCEDURE

- An amoebic liver abcess is a necrotic lesion and therefore the chances of detecting amoebae in it are quite low. A lack of trophozoites does not rule out *E. histolytica*.
- Amoebic cultures take several days and there is no guarantee that the parasite will adapt to culture conditions.

AUTHOR

Ian Crandall

REFERENCES

Garcia LS, Bruckner DA. Diagnostic Medical Parasitology. 3rd Edition.ASM Press, Washington DC. Pp676-688 1997.

Robinson, G.L. Trans. Royal Soc. Med. Hyg. 62:285 1968